**TERMOMETRO INFRARROJO
SIN CONTACTO DE PARED**
NON-CONTACT INFRARED THERMOMETER WALL

REF.: 1082

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- 1.** *Counting capacity: 999.*
- 2.** *Abnormal automatic alarm: leds flashing + acoustic signals.*
- 3.** *Automatic measurement: measuring distance 5 ~ 10 cm.*
- 4.** *Digital display.*
- 5.** *Power: by USB wire connected to an adaptor or by battery 18650 Li-ion (battery not included)*
- 6.** *Installation ways: by two screws, by two double-sided adhesive tapes, or by a bracket fixing.*
- 7.** *Environment temperature: 10 ~ 40°C (recommended 15 ~ 35°C).*
- 8.** *Infrared measuring range: 0 ~ 50 °C.*
- 9.** *Response time: 0,5 s.*
- 10.** *Input voltage: DC 5V.*
- 11.** *Weight: 350 g.*
- 12.** *Dimensions: 170 x 115 x 140 mm.*
- 13.** *Accuracy: $\pm 0,2$ °C (between 34 ~ 45°C)*

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1. *Standby: a red dot lights up in turn in the bottom of the display.*
2. *Normal temperature: flashing green the four leds + acoustic signal.*
3. *High temperature: flashing red the four leds + five acoustic signals.*
4. *Extreme temperatures (body mode):*
 - *Ultra low temperature: "Lo", flashing green the four leds + acoustic signal.*
 - *Ultra high temperature: "Hi", flashing red the four leds + five acoustic signals.*

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- 1.** Connect the thermometer to an adaptor DC 5V by the USB cable and turn on the upper button, position 1.
- 2.** Temperature measurement: place the surface to measure in front of the thermometer, distance 5 ~ 10 cm.
- 3.** Working modes: there is a small hole on the back side. Insert a screwdriver (diameter < 3 mm) to press an internal button.
 - (1) Temperature unit: Press the internal button to select °C or °F.
C: Celsius / F: Fahrenheit
 - (2) Working modes: Press the internal button for 3 s. to select the following working modes:
 - Sur: Surface object mode.
 - Bod: Body temperature mode.
 - Cou: Counting mode.
- 4.** Install and replace battery.
 - (1) Remove the four screws from the back side.
 - (2) Open the back base and install/replace the battery Li-ion model 18650.
- 5.** Installation ways.
 - (1) By two screws.
 - (2) By two double-sided adhesive.
 - (3) By a bracket fixing.

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IMPORTANT INFORMATION FOR CORRECT DISPOSAL OF THE PRODUCT

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Products must be kept clean and kept in a dry place. Don't put the electronic forehead temperature gun in a place with electric shock. Do not store the electronic forehead temperature gun in an extreme temperature environment above 55° C or below -10° C and humidity above 85%.

At the end of its useful life, the product cannot be disposed of as urban waste. It must be taken to a waste collection point, established by the competent authority or by suppliers specialized in this service. Recycling an electrical or electronic product in an adequate way avoids the negative consequences for the environment and for health derived from incorrect recycling and allows the raw materials that constitute it to be recycled, saving on natural resources and energy. As a reminder of the need to recycle electrical and electronic products properly, the product is marked with a crossed out container.

Edificio Genebre. Av. Joan Carles I, 46-48 - 08908 L'Hospitalet de Llobregat - Barcelona (Spain) Tel.: +34 902 504 203